

Using the LogTag®

- The LogTag® provided by ACT Health must remain with your government funded vaccines at all time
- Read and record the current/ minimum/ maximum temperature at least twice a day, first thing in the morning and at the end of the day
- Ensure the glycol bottle always remains in the fridge and connected to the LogTag
- You will not be able to disable the alarm, change settings or download data
- Contact the Vaccine Management Unit for help by calling (02) 5124 9800.

Reading the LogTag®

At a Glance

A ✓ will appear in the top left corner of the screen if readings have remained within the safe +2°C to +8°C range.

A X symbol in the top left corner, and an audible alarm indicates a cold chain breach.

Please contact the Vaccine Management Unit on 02) 5124 9800 immediately if this occurs, or you believe you have had a cold chain breach.

Reviewing daily readings

- The current temperature will display in the centre of the screen.
- Review stored data by pressing the “**REVIEW/ Mark**” button. Data will appear in the top of the screen.
- Each press of the button will step through Max and Min temperatures for each day, starting with today and moving backwards through the days. The displayed day will flash on the bottom bar of the screen.

For comprehensive instructions please see the ACT Health – Health Professions page – LogTag video at

<https://health.act.gov.au/services-and-programs/immunisation/health-professionals>.

